


NOVACOR Vista ECG Recorder

2 Channel LEAD PLACEMENT


Ch 1	V5 +	Red	5 th intercostal space, at the left of the axillary line.
	V5 -	Yellow	Right clavicle, at the sternal border
Ch 2	V1 +	White	4 th intercostal space, at the right of the sternal edge
	V1 -	Green	Left clavical, at the sternal border.
Ground		Black	Cavity of thorax, lower right chest wall.

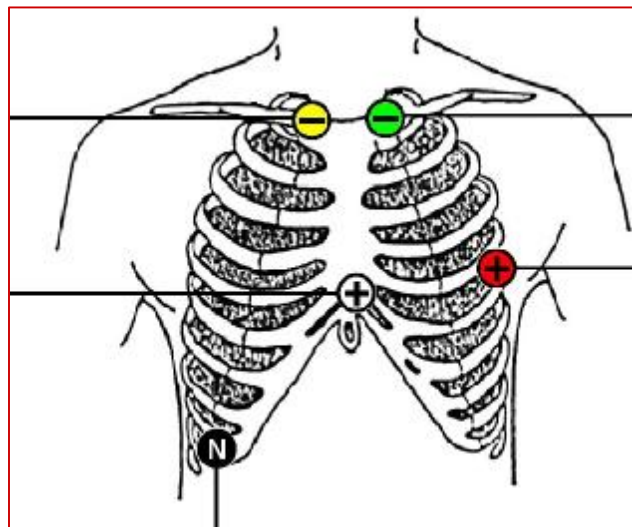

NOVACOR Vista ECG Recorder

3 Channel LEAD PLACEMENT

Ch 1	V5 +	Red	5 th intercostal space, at the left of the axillary line.
	CM -	Yellow	Common, upper part of sternum.
Ch 2	V1 +	White	4 th intercostal space, at the right of the sternal edge
	CM	Yellow	Common, upper part of sternum.
Ch 3	L5+	Green	L5
	CM-	Yellow	Common, upper part of sternum.
Ground		Black	Cavity of thorax, lower right chest wall.

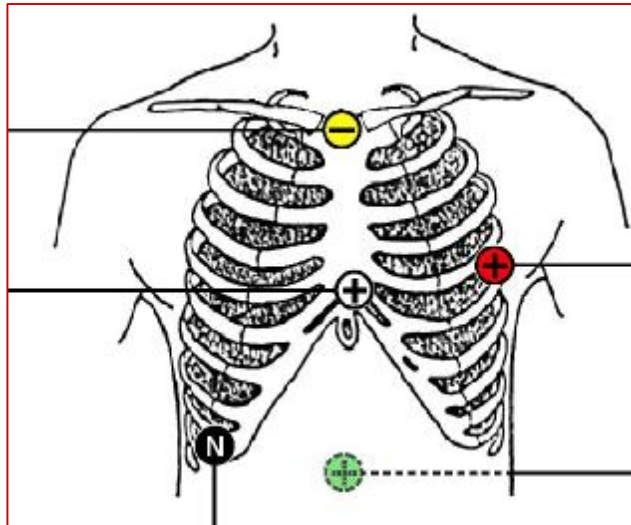

Alternatives for three anterior channels:

Green electrode in V4R, or green electrode in V4 and Red electrode in V6